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The screenshot displays the 'DensOffice - Systemdaten' application window. The main area is titled 'Schnittstellen' and contains various configuration options for external devices and programs. A sub-dialog box titled 'Konfiguration externe Programme' is open, showing a list of available interfaces. The 'VDDS-MMI:CEREC-VDDS' interface is selected and marked as 'aktiv'. The checkbox 'das Programm soll aktiv angesprochen werden' is checked. The 'OK' button is highlighted. Red arrows with numbers 3 through 9 indicate the sequence of steps for configuring the interface.

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The screenshot shows the DENSoffice software interface. The main window displays the 'DOX DENSoffice EXPERIENCED' logo and version information: 'DENSoffice Version 8.70 (5) - Limitiert (c) DENS GmbH 1994-2021'. The user interface includes a top menu bar with options like 'DensOffice', 'Ansicht', 'Videos', and 'Hilfe'. Below this is a toolbar with various icons for patient management, such as 'Neuer Patient', 'Benutzerwechsel', and 'Aufgabenverwaltung'. The main area shows a patient search dialog box titled 'DENSoffice - Patient suchen'. The dialog has a search input field containing the number '2'. Below the input field is a table with the following data:

Name, Titel	Vorname	Geb.-Datum	Status	Kasse	Behandler	Pat.-Nr.
1 Mustermann	Helmut	13.05.1952	M	AOK Nordost >Berlin	Dens	2
2						
3						
4						
5						
6						
7						
8						
9						
10						
11						
12						
13						
14						
15						
16						
17						
18						
19						
20						

At the bottom of the dialog box, there are buttons for 'OK', 'Spezielle Suche', 'Abbrechen', and 'Hilfe'. A red arrow labeled '10' points to the 'Externe Programme' button in the bottom toolbar, and another red arrow labeled '11' points to the 'OK' button in the dialog box.

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The screenshot displays the DENSoffice software interface. At the top, there is a menu bar with options like 'DensOffice', 'Ansicht', 'Videos', and 'Hilfe'. Below the menu is a toolbar with various icons for patient management, user actions, and system functions. The main window shows patient information for '(2) Mustermann, Helmut' and a 'Demo' watermark. A dialog box titled 'Auswahl externes Programm' is open, prompting the user to select an external program. The list of available programs includes 'DBSWin (Dürr Dental)' and 'VDDS-MMI:CEREC-VDDS', with the latter selected. Red arrows labeled '12' and '13' point to the selected program and the 'OK' button, respectively. The bottom of the screen features a function key bar (F1-F12) with corresponding icons and labels.

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(C) DENS GmbH 1994-2021